

PROSEPT® Impression S – Ready-to-use spray disinfectant for dental impressions and impression trays.



PROSEPT® Impression S

PROSEPT® Impression S is a spray disinfectant for the quick disinfection of dental impressions and impression trays. Thanks to its broad effective spectrum and its short contact time, PROSEPT® Impression S offers high efficacy while maintaining dimensional stability of impressions. Since PROSEPT® Impression S does not contain any aldehydes, no discoloration or blood and protein fixation will occur on the treated material. PROSEPT® Impression S is suitable for alginate, polyether, polysulfide, A-silicone, and C-silicone, and pleasantly fragranced.

Your Benefits

- ▶ **Broad effective spectrum**
Active against all bacteria including those causing tuberculosis, and viruses.
- ▶ **Short exposure time**
Active within 1 minute.
- ▶ **Will not alter the form or dimensional stability of dental impressions**
The disinfectant has no negative effect on the form or stability of dental impressions.

Effective Spectrum / Exposure Times

Bactericidal	60 seconds
Antibiotic-resistant bacteria	60 seconds
Tuberculocidal	60 seconds
Mycobactericidal	60 seconds
Yeasticidal	60 seconds
Antibiotic-resistant yeasts	60 seconds
Virucidal against enveloped viruses	15 seconds
Bovine viral diarrhoea virus (BVDV)	15 seconds
Coronavirus	15 seconds
Hepatitis B virus	15 seconds
Hepatitis C virus	15 seconds
Herpes simplex virus	15 seconds
Human immunodeficiency virus (HIV)	15 seconds
Influenza A virus	15 seconds
Vaccinia virus	15 seconds
Virucidal	60 seconds
Adenovirus	60 seconds
Norovirus	30 seconds
Poliovirus	60 seconds

Bactericidal (EN 13727:2012+A2:2015*, EN 14561:2006*) 60 sec; tuberculocidal (EN 14348:2005*, EN 14563:2008*) 60 sec; mycobactericidal (EN 14348:2005*, EN 14563:2008*) 60 sec; yeasticidal (EN 13624:2013*, EN 14562:2006*) 60 sec; enveloped viruses (EN 14476:2013+A1:2015*) 15 sec; virucidal (EN 14476:2013+A1:2015*) 60 sec

^ Tested under low organic load / * Tested under high organic load

Delivery Form

OD-071042 1 litre bottle

Easy to Use

- ▶ For your protection, wear disposable gloves and safety goggles.
- ▶ Completely moisten the dental impression and impression tray and leave disinfectant to act.
- ▶ Adhere to but do not exceed indicated contact time.
- ▶ After disinfection, rinse impressions thoroughly under running water of at least drinking quality for 10 seconds.
- ▶ Use on alcohol compatible materials only.
- ▶ A list of compatible and incompatible materials is available on our website.
- ▶ If you are uncertain as to the material composition of the medical device, contact the manufacturer before use.

Limitations

- ▶ FOR PROFESSIONAL USE ONLY. Not for use on invasive medical devices.
- ▶ Do not mix with other products.
- ▶ Make sure that all visible soiling has been removed before disinfection.

Compatibility

Acetal polymer	Compatible
Acrylic glass	Not compatible
Acrylonitrile butadiene styrene	Compatible
Alginate	Compatible
Aluminium	Compatible
Artificial leather	Compatible
A-silicone	Compatible
Chlorinated polyvinyl chloride	Compatible
C-silicone	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyether	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfide	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Rhodium	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

Relevant Physical and Chemical Properties

Colour:	Colourless
Physical state:	Clear, non-viscous liquid
Scent:	Of alcohol, lemon
pH-value:	Neutral
pH-value in aqueous solution:	Not applicable (ready-to-use solution)
Flammability:	The product is flammable
Relative density (g/cm ³):	0.92

Composition

100 g contains 20 g ethanol, 28 g 1-propanol, 0.056 g quaternary ammonium compounds, auxiliaries and water.

Shelf Life

Unopened container:	48 months
---------------------	-----------

The shelf life of the product is subject to correct storage and transportation conditions and the use of original containers. Keep containers tightly closed.

Danger

H226 Flammable liquid and vapour. H318 Causes serious eye damage. H336 May cause drowsiness or dizziness. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P261 Avoid breathing fume/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

Your Local Distributor

United Disinfectant Manufacturers AG
 Allmendstrasse 21
 8320 Fehraltorf
 Switzerland
 Telephone: +41 (0)55 511 03 33
 info@udm.swiss
 www.udm.swiss

