



PROSEPT[®] Ultima – Ready-to-use solution for the quick disinfection of non-invasive medical devices.









PROSEPT® Ultima

PROSEPT® Ultima is a ready-to-use disinfection and cleaning product for surfaces of non-invasive medical devices such as examination tables and dental treatment units. The gentle formulation of PROSEPT® Ultima is particularly well suited for alcohol-sensitive surfaces made of PVC, silicone, acrylic glass, or artificial leather. The low alcohol content prevents damage to the disinfected surfaces and reduces skin and breathing problems common with disinfectants containing a high alcohol percentage. PROSEPT® Ultima is free of aldehydes and phenols.

Your Benefits

- Aldehyde-, chlorine- and phenol-free
 Comprehensive disinfection with no health risks.
- Alcohol content of less than 23% The disinfectant is particularly suitable for use on sensitive surfaces such as artificial leather covers or acrylic glass.

Effective Spectrum / Exposure Times

Bactericidal	60 seconds
Antibiotic-resistant bacteria	60 seconds
Yeasticidal	60 seconds
Antibiotic-resistant yeasts	60 seconds
Virucidal against enveloped viruses	15 seconds
Bovine viral diarrhea virus (BVDV)	15 seconds
Coronavirus	15 seconds
Hepatitis B virus	15 seconds
Hepatitis C virus	15 seconds
Herpes simplex virus	15 seconds
Human immunodeficiency virus (HIV)	15 seconds
Influenza A virus	15 seconds
Vaccinia virus	15 seconds
Limited spectrum virucidal	5 minutes
Adenovirus	5 minutes
Norovirus	5 minutes

Bactericidal (EN13727:2012 + A2:2015*, EN 16615:2015^) 60 sec; yeasticidal (EN 13624:2013*, EN 16615:2015^) 60 sec; enveloped viruses (EN 14476:2013 + A1:2015*) 15 sec; limited virucidal (EN 14476:2013 + A1:2015^) 5 min

Delivery Forms

OD-032013	250 ml bottle	
OD-032016	1 litre bottle	
OD-032020	2 litre bottle	
OD-032025	5 litre canister	

[^] Tested under low organic load / * Tested under high organic load





Easy to Use

- For your protection, wear disposable gloves and safety goggles.
- Completely moisten and wipe the entire surface of the medical device
- Leave disinfectant to act and adhere to contact time.
- ▶ Use on alcohol compatible materials only.
- Before first time use, test material compatibility by applying the disinfectant to a small inconspicuous area.
- A list of compatible and incompatible materials is available on our website.
- If you are uncertain as to the material composition of the medical device, contact the manufacturer before use.

Limitations

- ► FOR PROFESSIONAL USE ONLY.
- Not for use on invasive medical devices.
- ▶ Do not mix with other products.
- Make sure that all visible soiling has been removed before disinfection.

Compatibility

Acetal polymer	Compatible
Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

Relevant Physical and Chemical Properties

Colour:	Colourless
Physical state:	Clear, non-viscous liquid
Scent:	Of alcohol
pH-value:	Neutral
pH-value in aqueous solution:	Not applicable (ready-to-use
	solution)
Flammability:	The product is flammable
Relative density (g/cm³):	0.97

Composition

100 g contains 7.5 g ethanol, 15 g isopropanol, 0.76 g quaternary ammonium compounds, auxiliaries and water.

Shelf Life

Unopened container: 24 months

The shelf life of the product is subject to correct storage and transportation conditions and the use of original containers. Keep containers tightly closed.

Danger

H225 Highly flammable liquid and vapour. H319 Causes serious eye irritation. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337 + P313 If eye irritation persists: Get medical advice/attention. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

Your Local Distributor

United Disinfectant Manufacturers AG Allmendstrasse 21 8320 Fehraltorf Switzerland

Telephone: +41 (0)55 511 03 33 info@udm.swiss

info@udm.swiss www.udm.swiss



All information provided is based on our current knowledge and does not constitute a legally binding assurance of specific product properties
© 2023 United Disinfectant Manufacturers AG. All rights reserved.