

PROSEPT® Wipes – Ready-to-use wipes for the quick, comprehensive, and residue-free disinfection of non-invasive medical devices.



PROSEPT® Wipes

PROSEPT® Wipes are pre-moistened, single-use wipes made of tear-resistant, non-woven material. PROSEPT® Wipes are perfectly suitable for the quick and easy disinfection and cleaning of surfaces of non-invasive medical devices such as examination tables, dental treatment units, and medical appliances. The carefully balanced formulation based on alcohols offers a broad effective spectrum and short contact times. The low content of volatile organic compounds (VOC) of less than 50% minimizes skin and breathing problems, provides excellent material compatibility, and results in residue-free drying. PROSEPT® Wipes are free of aldehydes and phenols. The convenient refill pouches ensure an economical and efficient disinfection.

Your Benefits

- ▶ **Ready-to-use disinfectant wipes**
The use of wipes eliminates the need for spray disinfection, which can cause respiratory tract, eye, and nose irritation.
- ▶ **Residue-free drying**
The disinfectant wipes leave no streaks and, therefore, make additional wiping redundant.
- ▶ **Dermatologically tested**

Effective Spectrum / Exposure Times

Bactericidal	60 seconds
Antibiotic-resistant bacteria	60 seconds
Tuberculocidal	60 seconds
Mycobactericidal	60 seconds
Yeasticidal	60 seconds
Antibiotic-resistant yeasts	60 seconds
Virucidal against enveloped viruses	15 seconds
Bovine viral diarrhoea virus (BVDV)	15 seconds
Coronavirus	15 seconds
Hepatitis B virus	15 seconds
Hepatitis C virus	15 seconds
Herpes simplex virus	15 seconds
Human immunodeficiency virus (HIV)	15 seconds
Influenza A virus	15 seconds
Vaccinia virus	15 seconds
Virucidal	60 seconds
Adenovirus	60 seconds
Norovirus	60 seconds
Poliovirus	60 seconds
Rotavirus	30 seconds

Bactericidal (EN 13727:2012+A2:2015*, EN 16615:2015*) 60 sec; tuberculocidal (EN 14348:2005*) 60 sec; mycobactericidal (EN 14348:2005*) 60 sec; yeasticidal (EN 13624:2013*, EN 16615:2015*) 60 sec; enveloped viruses (EN 14476:2013+A1:2015*) 15 sec; virucidal (EN 14476:2013*) 60 sec
^ Tested under low organic load / * Tested under high organic load

Delivery Forms

OD-041079	Refill pouch (14.5 x 20 cm), 120 pcs, classic scent
OD-041081	Refill pouch (14.5 x 20 cm), 120 pcs, classic scent
OD-041084	Refill pouch (14.5 x 20 cm), 120 pcs, lemon scent
OD-041092	Jumbo, Refill pouch (14.5 x 40 cm), 100 pcs, classic scent
OD-042080	Value, Refill pouch (14.5 x 20 cm), 250 pcs, lemon scent
OD-042084	Premium, Refill pouch (19.3 x 21 cm), 120 pcs, lemon scent
OD-091061	PROSEPT® Wipes Tub T1 (empty tub suitable for OD-041079, OD-041081, OD-041084)
OD-091060	PROSEPT® Wipes Tub T2 (empty tub suitable for OD-041092, OD-042080)
OD-091059	PROSEPT® Wipes Tub T3 (empty tub suitable for OD-042084)



Easy to Use

- ▶ For your protection, wear disposable gloves and safety goggles.
- ▶ Before use, cut open the refill pouch.
- ▶ Pull the first wipe at the heart of the roll through the star-shaped opening in the lid.
- ▶ Close the wipes box firmly after each use.
- ▶ Completely moisten and wipe the entire surface of the medical device.
- ▶ Leave disinfectant to act and adhere to contact time.
- ▶ Use on alcohol compatible materials only.
- ▶ Before first time use, test material compatibility by applying the disinfectant to a small inconspicuous area.
- ▶ A list of compatible and incompatible materials is available on our website.
- ▶ If you are uncertain as to the material composition of the medical device, contact the manufacturer before use.

Limitations

- ▶ FOR PROFESSIONAL USE ONLY.
- ▶ Not for use on invasive medical devices.
- ▶ Do not mix with other products.
- ▶ Make sure that all visible soiling has been removed before disinfection.

Compatibility

Acetal polymer	Compatible
Acrylic glass	Compatible
Acrylonitrile butadiene styrene	Compatible
Aluminium	Compatible
Artificial leather	Compatible
Chlorinated polyvinyl chloride	Compatible
Polyamide	Compatible
Polybutylene terephthalate	Compatible
Polycarbonate	Compatible
Polyetherimide	Compatible
Polyethylene	Compatible
Polyethylene terephthalate	Compatible
Polyphenylene oxides	Compatible
Polyphenylsulfone	Compatible
Polypropylene	Compatible
Polystyrene	Compatible
Polysulfone	Compatible
Polyvinyl chloride	Compatible
Polyvinylidene fluoride	Compatible
Rhodium	Compatible
Silicone	Compatible
Thermoplastic elastomer	Compatible
Thermoplastic polyurethane	Compatible
Ultra-high-molecular-weight polyethylene	Compatible

Relevant Physical and Chemical Properties

Colour:	Colourless
Physical state:	Liquid absorbed on fleece
Scent:	Of alcohol
pH-value:	Neutral
pH-value in aqueous solution:	Not applicable (ready-to-use wipes)
Flammability:	The product is flammable
Relative density (g/cm ³):	0.92

Composition

100 g moisturizing solution contains 31.3 g ethanol, 16.7 g isopropanol, auxiliaries and water.

Shelf Life

Unopened container:	36 months
---------------------	-----------

The shelf life of the product is subject to correct storage and transportation conditions and the use of original containers. Keep containers tightly closed.

Danger

H228 Flammable solid. H319 Causes serious eye irritation. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P233 Keep container tightly closed. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337 + P313 If eye irritation persists: Get medical advice/attention. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

Your Local Distributor

United Disinfectant Manufacturers AG
Allmendstrasse 21
8320 Fehraltorf
Switzerland
Telephone: +41 (0)55 511 03 33
info@udm.swiss
www.udm.swiss

